

Emergency Obstetrics
Simulation Suite

Emergency Hysterectomy
Simulator

Product Number: OB-200-2000
1022564

Table of Contents

IMPORTANT INFORMATION.....	3
Introduction	4
Skills Taught	4
Items Included.....	4
Unpacking and Assembly	5
Directions for Use	5
Preparing the Simulator	5
Installing the Uterus into the Simulator	5-6
Install the Intestines and Foam Abdomen.....	7
Infusing with Fake Blood	8
Care and Maintenance	9
Replacement Parts	9
Customer Support	9

Important Information

GENERAL WARNINGS

Warning: A poorly lubricated uterus may damage the Simulator during operation.

Only use the lubricant and simulated blood supplied with this product. Use of non-recommended fluid products may weaken or damage the simulator.

Lubrication is key to the usage of this product; inadequate lubrication could lead to tears or other damage to silicone parts. Apply at least 2-3 squirts of lubrication to the uterus before use.

It is vitally important that after each training session all lubrication is cleaned with a warm damp cloth. You can also use baby wipes to wipe away any excess lubricant.

Warning: Sharp objects, excessive force, or using non-OEI products can scratch, tear or damage the simulator.

Gloves should always be worn, and all jewelry items should be removed. Sharp surfaces such as finger nails and rings can sometimes lead to damage of silicone parts.

WARRANTY INFORMATION

WARRANTIES. EXCEPT AS OTHERWISE SET FORTH HEREIN, OPERATIVE EXPERIENCE, INC. ("OPERATIVE EXPERIENCE") PRODUCTS ARE SOLD ONLY WITH THE 1-YEAR LIMITED WARRANTY DESCRIBED BELOW. THIRD-PARTY PRODUCTS ARE EXPRESSLY EXCLUDED FROM THE BELOW LIMITED WARRANTY, AND ARE SUBJECT ONLY TO THOSE WARRANTIES EXTENDED BY THE APPLICABLE THIRD PARTY MANUFACTURERS OF SUCH PRODUCTS, IF ANY. EXCEPT WHERE PROHIBITED BY LAW OR AS OTHERWISE PROVIDED HEREIN, OPERATIVE EXPERIENCE HEREBY EXPRESSLY DISCLAIMS ALL WARRANTIES EITHER EXPRESS OR IMPLIED, INCLUDING WITHOUT LIMITATION, THE IMPLIED WARRANTIES OF MERCHANTABILITY, FITNESS FOR A PARTICULAR PURPOSE, NON-INFRINGEMENT, AND ANY WARRANTIES ARISING FROM A COURSE OF DEALING, COURSE OF PERFORMANCE, OR USAGE OF TRADE. OPERATIVE EXPERIENCE PERSONNEL, SUPPLIERS, AND LICENSEES ARE NOT AUTHORIZED TO ALTER THESE WARRANTIES.

Operative Experience warrants that its products ("Products") are free from defects in materials and workmanship, subject to specific limitations and conditions set forth below, for a period of one (1) year from the date of the original purchase by the first end-user (the "Warranty Period"). Claims of defect brought after the Warranty Period will not be covered by this limited warranty. In order to bring a claim under this limited warranty, the buyer must notify Operative Experience by email to info@operativeexperience.com, describing the defect in reasonable detail. If Operative Experience determines that such defect may be covered by this limited warranty, Operative Experience will issue a return material authorization ("RMA") to the buyer, and the buyer shall return the Product in question to Operative Experience using the instructions provided by Operative Experience. If Operative Experience, in its reasonable discretion, determines that a Product has a defect in materials or workmanship, Operative Experience shall, at its option, repair or replace the defective Product at no cost to the buyer. If identical materials are not available at the time of repair or replacement, Operative Experience may, at its option, substitute materials of comparable quality or refund buyers money. The Products are intended only for use in medical training, and this limited warranty shall not cover any Product used for commercial or other non-medical training purposes. Any alteration by buyer of a Product shall render this limited warranty void with respect to the altered Product. This limited warranty extends only to the original buyer and shall not cover defects or damages resulting from (i) accident, misuse, abuse, tampering, neglect, unusual physical stress, alteration, or improper operation or handling of the Product; (ii) incorrect or inadequate maintenance, cleaning, or care of the Product; or (iii) normal wear and tear of the Product. Operative Experience will not be responsible under this limited warranty or otherwise for defects or damage caused by any ancillary equipment not furnished by Operative Experience which is attached to or used in connection with the Products, and all such equipment is expressly excluded from this limited warranty. This limited warranty does not extend to consumable parts, such as batteries, or any third-party products that buyer may use or purchase in conjunction with the Products.

INTRODUCTION

The Emergency Hysterectomy Simulator is an open abdomen simulator supporting hemorrhage control and fully operable removal of the uterus.

Skills Taught

- Fully operable removal of the uterus
- Teamwork and human factors training emergency Caesarean section

Items Included

1 Simulator Base Assembly Part Number: OB-200-2000	Qty: 1	5 Lubricant, 16oz/473ml bottle Part Number: 220007	Qty: 1
2 Emergency Hysterectomy Uterus Part Number: OB-200-2201	Qty: 10	6 Intestines Part Number: OB-050-0300	Qty: 1
3 Open Abdomen Part Number: OB-050-2500	Qty: 1	7 Spare Hardware Kit Part Number: OB-300-2400	Qty: 1
4 Artificial Blood Part Number: OB-200-0700	Qty: 1	8 Artificial Blood Infusion Injector Part Number: OB-200-0750	Qty: 1



Unpacking and Assembly

NOTE: Use caution when opening the shipping carton, as this is the preferred storage container. Damage to the container could result in an inability to store it properly, leading to damage to the simulator and accessories.

C-celia comes in a specially designed carton with cutouts in the internal foam that provide for placement of all parts and accessories. Keep all original shipping materials, including boxes.

When unpacking C-Celia for the first time, careful use of a box cutter protects both the packaging and the product.

It is recommended that you reserve a space approximately 24in/60cm by 72in/183cm into which you can place the shipping carton for storage. Remove each item in the carton and verify, on the shipping paperwork, that everything is included.

Directions for Use

Preparing the Simulator

Prior to simulating a delivery, the simulator needs to be fully lubricated before use.

Using the lube provided, generously apply lube to the abdominal cavity and apply to all areas of the cavity using your hands.



Installing the Uterus Into the Simulator




Using the provided lube generously lubricate the entire uterus and attachment.




Carefully slide the uterus into the pelvis and place the provided anchor clip into the guide holes in the uterus base.




3
Make sure the anchor clip is fully inserted into the uterus base.

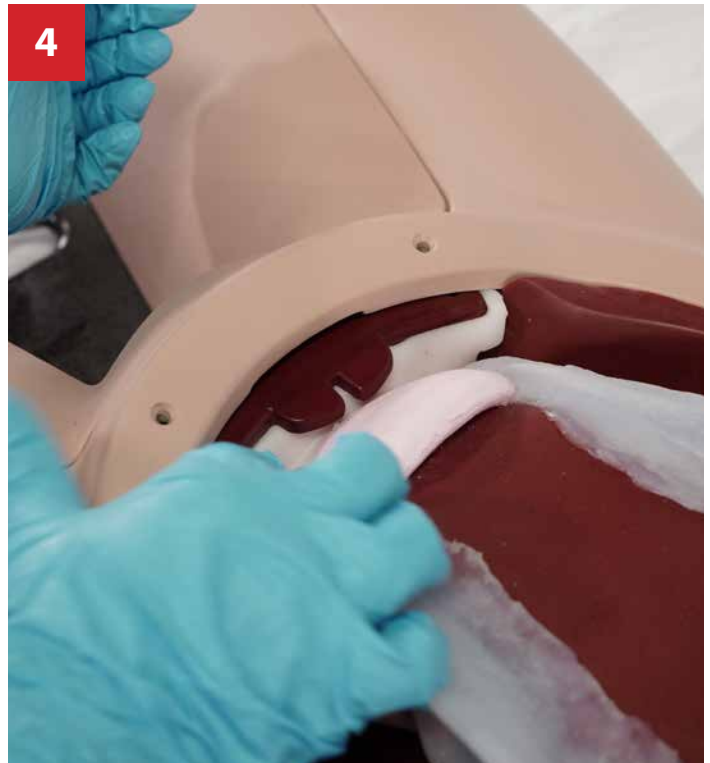

4
Once fully inserted, push the anchor clip down and under the trim of the simulator base until the screw holes align.

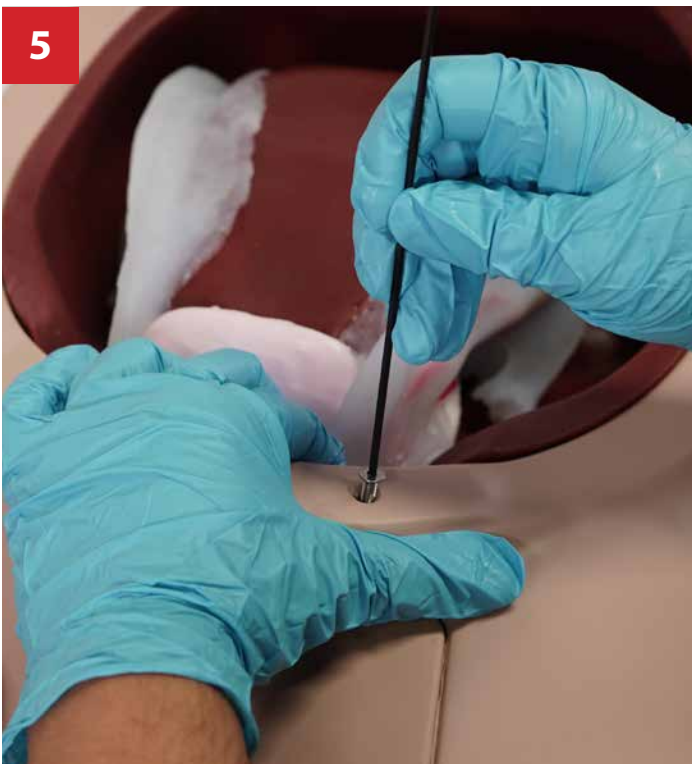

5
Screw the clip to the simulator base using the provided hardware.

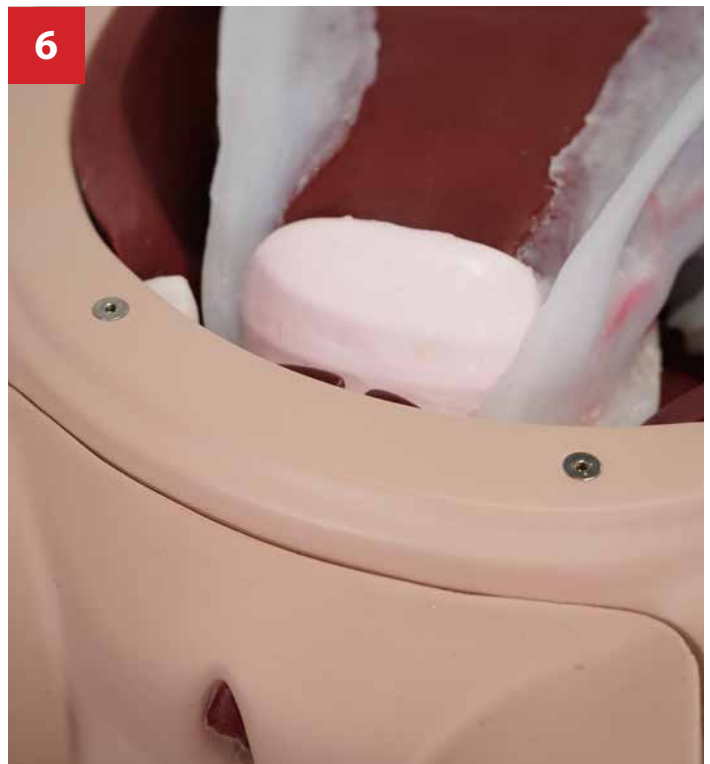


6
Do not overtighten or cross thread.

Install Intestines and Open Abdomen



Lubricate the intestines and place in the abdomen cavity under the uterus.

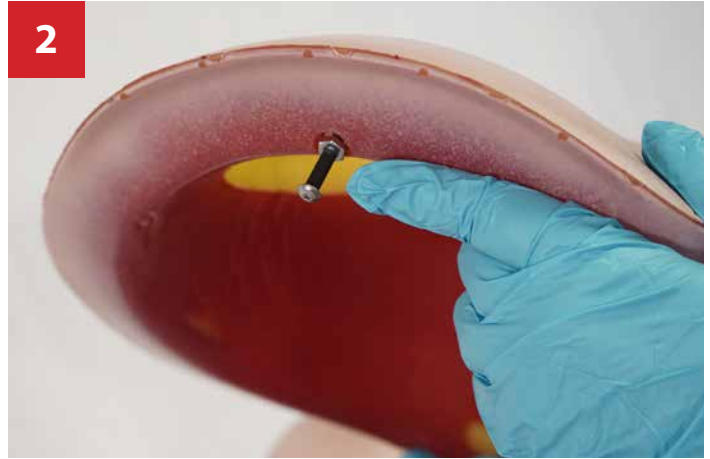

To attach the open abdomen, first locate the locking clip on the underneath of the abdomen.



Holding the abdomen at an angle, slide the locking clip into the small opening on the uterus anchor clip to hold it in place at the front.



Feed the velcro strip at the rear of the abdomen into the slit in the abdomen base.



Tighten and secure the abdomen with the velcro strip underneath the simulator.



Close door of the simulator.

Infusing with Fake Blood



Fill the injector with fake blood. Inject the blood in the upper third of the uterus. Repeat 8-10 times.



Massage the uterus to distribute the blood through the tissue.

The simulator is now ready for use.

Care and Maintenance

Surface

Use only on clean surface. Avoid felt tipped markers, ink pens, acetone, iodine or other staining products and avoid placing the module/manikin on newsprint or inked lines of any kind.

Lubrication

The provided lubricant should be applied liberally to the inside of the abdominal cavity.

Storage

All fluid should be removed from the abdominal cavity before storage. All surface areas should be cleaned and dried before storage.

Cleaning

Simulators should be wiped clean with a solution of mild soap and water. Clean with mild soap and water; do not submerge simulator, module or parts in cleaning fluids or water. The entire model can be rinsed using a hose. Parts should be drained and air-dried thoroughly before storage and disinfected when needed.

Replacement Parts

Description	Part Number
Shell door	OB-100-2005
Uterus Holder Clip	OB-300-2008
C-Hyst Uterus	OB-200-2201
Open Abdomens	OB-050-2500
Lubricant	220007
OB Spare Parts Kit	OB-300-2400



MORE INFORMATION AVAILABLE ONLINE AT 3BSCIENTIFIC.COM



3B Scientific GmbH • Ludwig-Erhard-Straße 20 • 20459 Hamburg • Germany • 3bscientific.com

Phone: +49 40 73966-0 • Fax: +49 40 73966-100 • E-mail: info@3bscientific.com