<i>English</i>	4
<i>Deutsch</i>	6
<i>Español</i>	8
<i>Français</i>	12
<i>Português</i>	14
<i>Italiano</i>	16
<i>Japan</i>	18
<i>Русский</i>	20
中文	22

3B Baby Care Model

English

Dear Customer,

Congratulations! In choosing the 3B Baby Care Model you decided for a high quality product. To keep the 3B Baby Care Model in top condition we have the following tips for you:

- Please do not expose the model to extraordinary thermic stress for prolonged periods (e.g. direct sunlight).
- When necessary, the model may be cleansed with a mild soap solution. Any persistent stains, except in the painted area of the head, may be cleansed additionally with a cleaning alcohol. However, please do not use solvents!
- Colored solutions, disinfection fluid containing iodine and the use of marker pens (permanent or otherwise) can lead to a permanent discoloration of the skin.
- To get as close as possible to a realistic weight and feel, the 3B Baby Care Model is filled with a gel. Please do not use sharp objects during training which could puncture the skin.
- If you need to use lubricants at any time, please ensure only water soluble ones are used. Do not use fat or oil based lubricants.

The 3B Baby Care Model is only suitable for the training possibilities mentioned in this brochure. It is not a toy for children.

In case of questions or complaints we kindly ask you to mention the serial number of your 3B Baby Care Model which can be found at the sole of the foot.

We wish you an interesting and successful training together with the 3B Baby Care Model!

By the way, at www.3bscientific.com you can always get the most up-to-date news on 3B products and developments.

General information

Size: 51 cm
Weight: 3.000 g
Head circumference: 33 cm
Supplied with carrying case

3B Baby Care Model

English

The 3B Baby Care Model was developed in close cooperation with pediatric nursing professionals. This true-to-life model of a newborn male baby is suitable for a variety of different training procedures:

Changing the baby's diaper

All common changing techniques using disposable or cloth diapers may be practiced. Baby oil, cream, or powder should not be applied.

Feeding and administration of medicine

Positioning of the baby for breast and bottle feeding as well as the administration of medication with a dropper can be practiced. Please only use water to keep the inner connection to the urethra clean.

Obtaining urine specimens

A connection between the mouth and the urethra allows the administered liquid to be excreted after a short period. To simulate urine specimen collection, a standard urine collection bag may be used.

Dressing and undressing

The dressing and undressing of a baby can be practiced with standard newborn clothing.

Picking up, carrying, and positioning of a baby

The natural weight of the baby and the flexibility of the arms and the legs allows all movements necessary for picking up, carrying, and positioning a baby.

Washing, cleaning of mouth, eyes, ear and nose

To practice the correct positioning of the baby during bathing, the 3B Baby Care Model may be placed in warm water (up to max. 40°C). Mouth, eyes, ears and nose may be also cleansed with a gauze pad or cotton swab. The use of a special bulb syringe for nose cleaning is also possible. In order to demonstrate the prevention of the intrusion of water into the ear, the flaps at the entrance to the ears may be closed with light pressure from the thumb and middle finger.

Taking a rectal temperature

Standard rectal thermometers may be used to simulate the taking of a rectal temperature.

Examination of fontanelles and testicles

The soft skin of the head allows palpation of the anterior and posterior fontanelles. A palpation of the testicles can also be practiced.

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3B Scientific GmbH